



PL 94-104 Łódź, ul. Obywatelska 187/189  
tel. +48 42 640 48 70, fax +48 42 688 33 84

## Instructions for use of ESTETIC ORT KOLOR

liquid dyes 



Please read these instructions carefully before using the product. The instruction is available on the website [www.wiedent.com.pl](http://www.wiedent.com.pl) in the tab: products / acrylic materials / Estetic Ort Kolor.

### Intended use

The Estetic Ort Kolor liquid dyes is a class I medical device intended for making coloured tooth-correction apparatuses.

### Description

The liquid dyes (blue, red, yellow, orange) made of ESTETIC ORT liquid are designed for dyeing the transparent material ESTETIC ORT, which enables making coloured tooth-correction apparatuses. The liquid dyes can be mixed to obtain other colours than the basic ones.

### Raw materials used for production

The ESTETIC Ort Kolor liquid dyes consist of a liquid containing methyl methacrylate monomer, ethylene glycol dimethacrylate and N, N dimethyl-p-toluidine and organic dyes.

### Direction for use

To obtain a colour, mix the liquid dyes in the proportion of 3-5 drops to 5 ml of the liquid in a pot used for this purpose; a more intense colour can be obtained by adding a bigger quantity of dye. The solution should be used in according to the instructions for use of ESTETIC Ort.

### Warnings and recommended precautions

The liquid is inflammable and also irritant. Avoid inhaling the vapours and prevent any contact with the skin and eyes. In case of contact, rinse thoroughly with water and, if any, contact your ophthalmologist. Don't dispose of the remainder of the liquid into the sewer. Dental technicians are warned against inhaling dust while carrying out prosthesis work. When engaged in this work, using of a dust-mask and a fume cupboard is recommended. The manufacturer recommends the following guidelines to be included in instructions for the patients:

- orthodontic apparatuses should be washed with a tooth-brush and toothpaste or any other agents especially designed for the care of such products,
- don't soak them in heavily alkali or acid substances in order to avoid damage of surface,
- damaged orthodontic apparatuses should be repaired only by a dental technician or a dentist,
- the adjustment of orthodontic apparatuses should be regularly checked.

### Packaging

Sets:

- 4 x 12,5 ml liquid (blue, red, yellow, orange).

### The storage and transport conditions

The ESTETIC Ort Kolor liquid dyes should be stored in tightly-closed containers in well-ventilated rooms. Keep out of the reach of children. Avoid direct sunlight and heat sources during storage and transport. They must be used within 5 years from the date of manufacture.

### Legal basis

The ESTETIC Ort Kolor liquid dyes meet the requirements of Regulation (EU) 2017/745 of the European Parliament and of the Council.

### Symbols on the packaging



Manufacturer



Consult instructions for use



Mark of product conformity with requirements of Regulation (EU) 2017/745 of the European Parliament and of the Council



Temperature limit



Keep away from sunlight



The product is a medical device



Batch code



Use-by date

Date of issue: 17.08.2022

D 3322-20