



PL 94-104 Łódź, ul. Obywatelska 187/189  
tel. +48 42 640 48 70, fax +48 42 688 33 84

Instructions for use of wax bite rims



Please read these instructions carefully before using the product. The instruction is available on the website [www.wiedent.com.pl](http://www.wiedent.com.pl) in the tab: products/ dental waxes / wax bite rims.

#### Intended use

The wax bite rim is a class I medical device intended for making dentures.

#### Description

Wax bite rims is characterized by very good plasticity achieved by using beeswax and natural resins in its production.

#### Raw materials used for production

The product consists of natural waxes, synthetic paraffins and paraffin dye.

#### Direction for use

Before use plastify the rim of wax by dipping it in warm water or holding over the flame of a burner. The rim of wax, prepared in above mentioned method can use in prosthodontic.

#### Warnings and recommended precautions

The shelf life of the product depends on the way it is stored. Wax bite rims stored incorrectly can lose its plasticity, color and form over time.

#### Packaging

The wax bite rims are offered in packs of 48 pieces.

#### The storage and transport conditions

Keep packaged, in a dry place, protected from the sun and far from any source of heat. Use by: 3 years from the date of production.

#### Legal basis

Dental wax meets the requirements of Regulation (EU) 2017/745 of the European Parliament and of the Council.

#### Symbols on the packaging



Manufacturer



Consult instructions for use



Mark of product conformity with requirements of Regulation (EU) 2017/745 of the European Parliament and of the Council



Temperature limit



Keep away from sunlight



The product is a medical device



Batch code



Use-by date

Date of issue: 04.07.2023

D 2723-1