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**Instructions for use of ALMAMISS acrylic teeth**

Please read these instructions carefully before using the product. **Intendend use:** ALMAMISS acrylic teeth are a class I medical device intended for the making of removable dentures. **Description:** ALMAMISS acrylic teeth are made of three layers: the neck, dentine and enamel, mimicking the structure of natural teeth. The posterior teeth have true anatomic features and the design of their cusps enables multipoint contact. Thanks to the use of a double-cross-linked IPN polymer, ALMAMISS teeth are highly abrasion and impact resistant. They are produced in 42 moulds, upper and lower anterior and upper and lower posterior, in universal V (16 shades) A1 - D4 shade scale. The 9 selected moulds are produced in two bleached shades of the universal V scale - 0M1 and 0M3. Our extensive range enables meeting all patient's needs both in terms of shape and shade. ALMAMISS acrylic teeth meet the requirements of the international standards EN ISO 22112 and EN ISO 7491. ALMAMISS acrylic teeth, like the natural teeth, have a fluorescent effect. **Raw materials used for production:** ALMAMISS acrylic teeth are made of powders - polymers based on methyl methacrylate and allyl methacrylate containing benzoyl peroxide (PMMA type IPN) and pigments, as well as liquids - methyl methacrylate monomer (MMA) and 1,4 butandiol dimethacrylate (1,4 BDDMA). **Direction for use:** In order to ensure a good bonding of the tooth to the denture base ALMAMISS posteriors have a deep transversal relief. **In case of grounding down of this relief making of deep retention is recommended to provide a good bonding of the dentin which is very hard to the denture base.** Before setting the teeth in the denture plate **any wax residues should be removed with boiling water and the teeth degreased with an acrylic monomer.** The teeth can withstand temperatures of up to 100°C and after the polymerization process bonding them with the denture plate they retain a glossy surface, which can be easily polished.

Warnings and recommended precautions:

- danger to the prosthetist may be inhalation of dust while grinding teeth,
 - It is recommended to use an exhaust hood and a dust mask during these works.
- The manufacturer recommends that the following guidelines be included among the instructions for the patient:
- acrylic teeth dentures used by the patient inconsistently with their intended use may pose a threat to their health because, just like natural teeth, they are exposed to surface abrasion, chipping or complete breakage from the prosthesis during use,
 - removable dentures made of acrylic teeth should be removed from the oral cavity for a night break,

- dentures should be cleaned with a tooth brush using an ordinary tooth paste or other substances specially formulated for denture cleansing,
- in order to avoid any damage to its surface dentures should not be immersed in any strongly alkaline or acidic substances,
- damaged dentures should only be repaired by a prosthetist or dentist,
- the fit of dentures should be periodically checked (especially in the case of fixed dentures),
- dentures not suitable for further use do not require specific treatment, are recyclable and can be disposed of with municipal waste.

The storage and transport conditions: ALMAMISS acrylic teeth should be stored a clean and dry place. During storage and transport, the temperature should be kept between 3 and 30°C and direct sunlight and heat sources should be avoided. They must be used within 10 years from the date of manufacture mainly due to possible deterioration of the packaging. **Legal basis:** ALMAMISS acrylic teeth meet the requirements of Regulation (EU) 2017/745 of the European Parliament and of the Council. Date of issue: 09.05.2022

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GB

**Instructions for use of ALMAMISS
acrylic teeth** 1639

ALMAMISS acrylic teeth have been designed for use in all types of prosthetic dentures. The teeth consist of three layers (neck, dentine and enamel), imitating the structure of natural teeth and like the natural ones have a fluorescent effect. ALMAMISS anterior teeth have differentiated final surface with transparent and luminous enamel.

The posterior teeth have true anatomic features and the design of their cusps enables multipoint contact. Thanks to the new enamel formula based on double cross-linked IPN type polymers and to the enhancement of mechanical properties of the dentin, ALMAMISS teeth are highly abrasion and impact resistant. They are manufactured in all universal V shades (A₁-D₄). Selected moulds are available in two bleached shades OM1 and OM3. In order to ensure a good bonding of the tooth to the denture base ALMAMISS posteriors have a deep transversal relief. **In case of grounding down of this relief making of deep retention is recommended to provide a good bonding of the dentin which is very hard to the denture base.** Before setting the teeth in the denture plate any wax residues should be removed with boiling water and the teeth degreased with an acrylic monomer. The teeth can withstand temperatures of up to 100°C and after the polymerization process bonding them with the denture plate they retain a glossy surface, which can be easily polished. ALMAMISS teeth meet the requirements of the international standards EN ISO 22112 and EN ISO 7491.

There is a potential hazard, however, in breathing in the dust by the technician during grinding or polishing work. It is therefore recommended that a suitable extraction system is used and a dust mask worn during such work.

ALMAMISS teeth should be stored in their original packaging in a clean, dry place away from the sunlight or a source of heat. They must be used within ten years from the date of manufacture mainly due to possible deterioration of the packaging.

The manufacturer recommends that the following guidelines be included among the instructions for the patient:

- Acrylic teeth dentures should be cleaned with a tooth brush using an ordinary tooth paste or other substances specially formulated for denture cleansing.
- Damaged dentures should only be repaired by a suitably qualified person (dentist, dental technician or a denturist).
- In order to avoid any damage to its surface dentures should not be immersed in any strongly alkaline or acidic substances.
- Patients are advised to have their prosthetic dentures checked periodically by a dentist.

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