



PL 94-104 Łódź, ul. Obywatelska 187/189
tel. +48 42 640 48 70, fax +48 42 688 33 84



Instructions for use of occlusal platewaxes



Application:

- to use by dentists for establishing the position of the patient's upper jaw in relation to the lower jaw.

Properties:

- very good plasticity achieved by using beeswax and natural resins in their production.

Direction for use:

Before use, plastify the platewax by dipping it in warm water or holding over the flame of a burner. Put the wax, prepared in above mentioned method, into the patient's mouth. Mould the edges protruding from the lips and cheeks side by pressing them to the teeth.

Storage:

Keep packaged, in a dry place, protected from the sun and far from any source of heat.

Best before: 3 years from the date of production.