



PL 94-104 Łódź, ul. Obywatelska 187/189  
tel. +48 42 640 48 70, fax +48 42 688 33 84

Instructions for use of occlusal platewaxes  

Please read these instructions carefully before using the product. The instruction is available on the website [www.wiedent.com.pl](http://www.wiedent.com.pl) in the tab: products/ dental waxes / occlusal platewaxes.

#### Intended use

The platewax is a class I medical device intended for establishing the position of the patient's upper jaw in relation to the lower jaw.

#### Description

The platewax is characterized by very good plasticity achieved by using beeswax and natural resins in its production.

#### Raw materials used for production

The product consists of natural waxes, synthetic paraffins and paraffin dye.

#### Direction for use

Before use, plastify the platewax by dipping it in warm water or holding over the flame of a burner. Put the wax, prepared in above mentioned method, into the patient's mouth. Mould the edges protruding from the lips and cheeks side by pressing them to the teeth.

#### Warnings and recommended precautions

The shelf life of the product depends on the way it is stored. Platewaxes stored incorrectly can lose its plasticity, color and form over time.

#### Packaging

Occlusal platewaxes are offered in packs of 21 pieces.

#### The storage and transport conditions

Keep packaged, in a dry place, protected from the sun and far from any source of heat. Use by: 3 years from the date of production.

#### Legal basis

Dental wax meets the requirements of Regulation (EU) 2017/745 of the European Parliament and of the Council.

#### Symbols on the packaging



Manufacturer



Consult instructions for use



Mark of product conformity with requirements of Regulation (EU) 2017/745 of the European Parliament and of the Council



Temperature limit



Keep away from sunlight



The product is a medical device



Batch code



Use-by date

Date of issue: 04.07.2023

D 2723-2